

Ringbio

Ringbio

Swine Pregnancy Rapid Test Kit

Fast Simple Reliable

Ring Biotechnology Co Ltd

info@ringbio.com


Ringbio Pig Pregnancy Rapid Test Kit

In each kit

- *50 test cards in foil pouch*
- *50 plastic pipet*
- *Kit instruction*

Sample

- *Fresh pig urine from sows 22-32 days after AI or mating*
- *Bring sample to room temp. before testing*

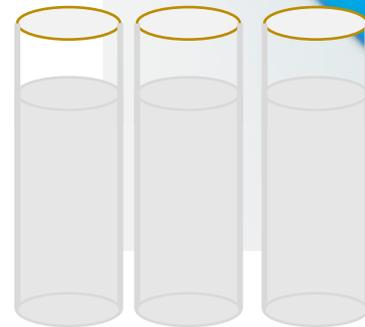


How to run the pig pregnancy test kit



Sample





Plastic pipet

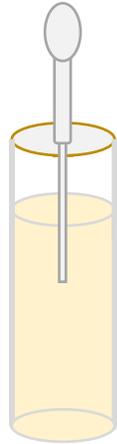


Pipetting Sample

Add 6 drops into the sample well on the card

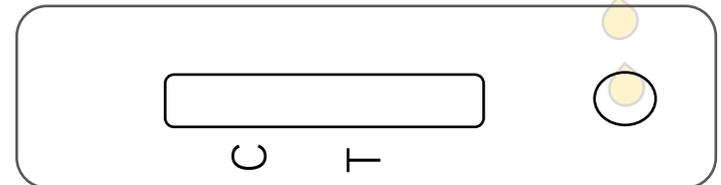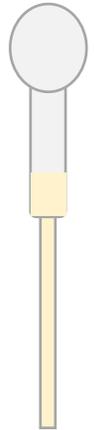

Sample flow on the card from sample well to test zone

*Wait for 10min
Then determine the result*

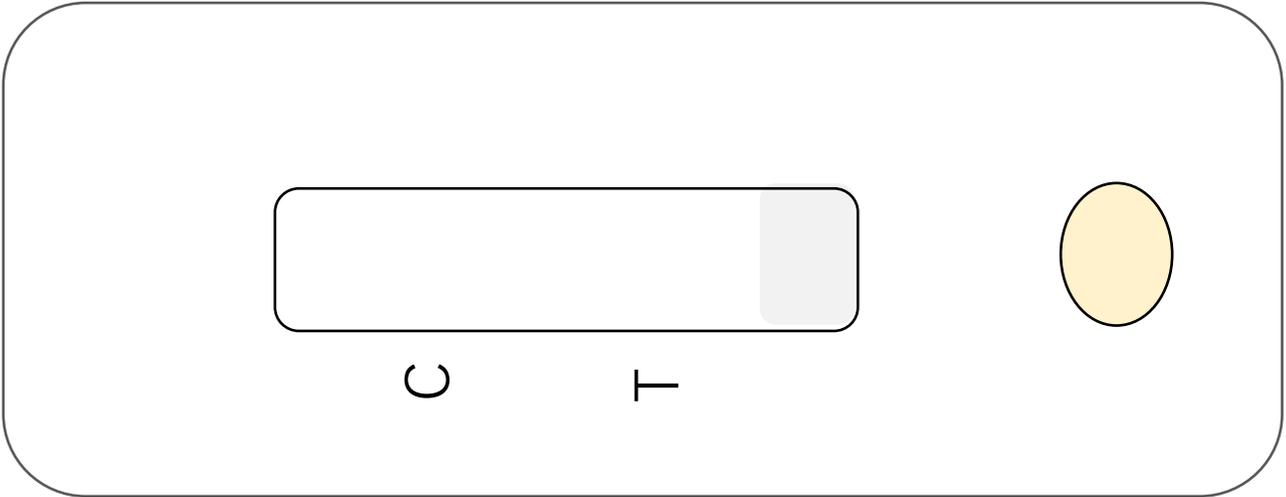


Determine the result

Compare the color of Line C & Line T


Positive: $C \geq T$
(including No Line T)

Negative: $C < T$


Invalid: No Line C

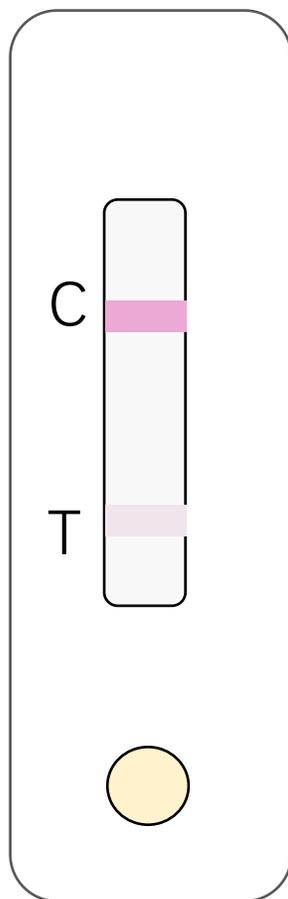


Positive

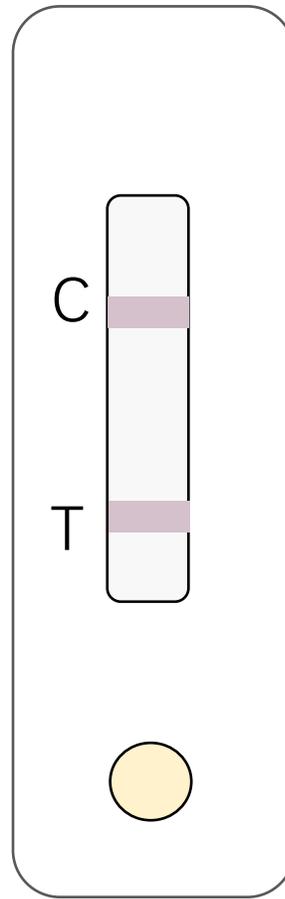

Positive

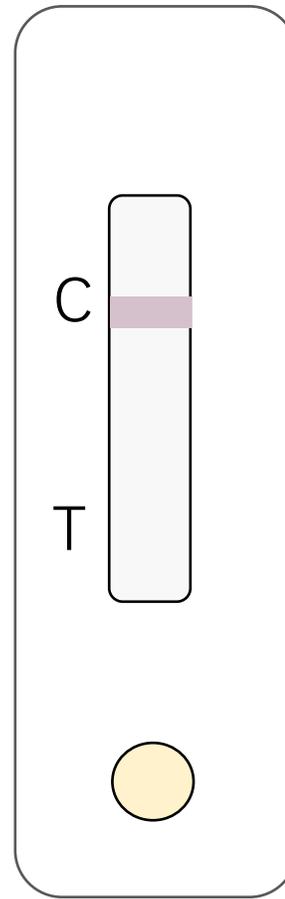

Positive

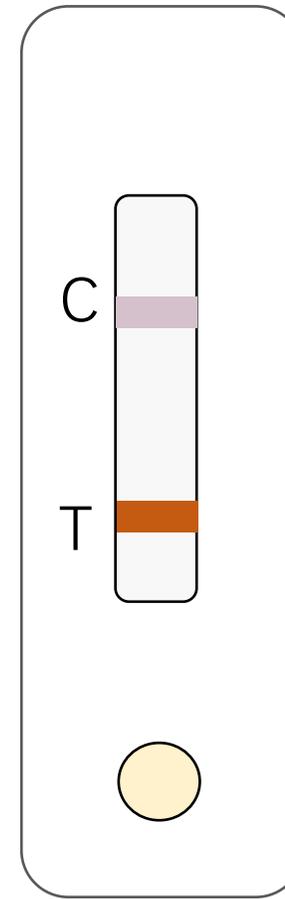

Negative

Caution

- *This kit is only for sows after breeding or AI,.*
- *This kit can not be used with boar or open pig urine.*
- *This kit can not detect pig pregnancy 48-62days after breeding or AI due to the hormone concentration change.*

Ringbio

Contact us

www.ringbio.com

info@ringbio.com